



INTRUSAFE

CARE + MAINTENANCE PRODUCTS

Instrument Tip Protectors

The Instrument Tip Protectors are intended to protect surgical instruments through processing and use. Guards are to be placed over the clean instrument and can be left on during handling, storage and sterilization.








They are approved for sterilization via the following methods:

- Gravity Steam Autoclave at 121° C for 30 minutes
- Pre-Vac Steam Autoclave at 132° C for 3 minutes
- Ethylene Oxide (EO)
- STERRAD® Standard Cycle (approved for web guards only)

***Note:** *these are minimum cycles*

Web Guards are designed to be used exclusively with low heat, plasma sterilization processes, such as the Sterrad Sterilization System.

Symbol Reference Key

	Consult Instructions For Use
	Quantity
	Catalog Number
	Lot Number
	Non-Sterile
	For Us Audiences Only
	Date Of Manufacture



815 Vikings Parkway, Suite 100
St. Paul, MN 55121 | USA

P : 1-888-229-2875 | 651-789-3939
F : 1-888-229-1941 | 651-789-3979

www.summitmedicalusa.com