



# INSTRUSAFE

## INSTRUMENT COLLECTION CONTAINER

### INSTRUCTIONS FOR USE

#### Indications for Use:

The container is used to collect used instruments and materials for transportation. The container can be used in a washer-disinfector and withstands repeated washing. The container can be autoclaved only if the base and cover are separated. Do not autoclave assembled. Do not use as a sterile barrier.

#### Material:

The container is made of high temperature plastic and is highly durable against heat and chemicals.

#### Construction:

The container consists of a lid, gasket, and base. The detachable lid can be removed to help facilitate cleaning. The silicone gasket in the lid creates an air tight seal and can be easily replaced. The container base holds the instruments and materials for transportation.

#### Lid Removal:


Unlatch the lid from the base and open until it is vertical. Slide the lid with pins out of the hinges in the base for easy cleaning.

To reassemble, align pins in lid with hinges while lid is vertical, sliding pins into hinges until fully engaged. If the lid is lowered before properly in place, the hinges, pin or lid could bend or crack.

#### Container Use:

1. Set container on stable surface
2. Add cleaner to container
3. Place instruments / materials in container for transportation
4. Close lid by fastening the locks
5. Transport container
6. Open lid slowly to remove instruments / materials
7. Autoclave sterilize or washer-disinfect the instruments / materials per their IFU

*Pin Shown Disengaged*

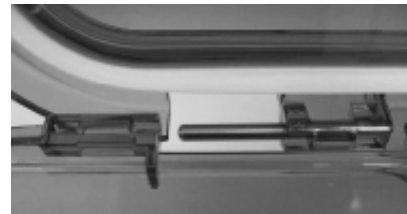

#### Warnings:

1. Do not drop. This may cause breakage to the container and materials inside or container leakage
2. Do not use if container is cracked or broken
3. If the latches on the lid are not fastened, leakage or splashing can occur
4. Do not carry container by the latches. This can cause the lid to come off and cause leakage, splashing damage or injury
5. Do not shake the container with liquids inside. This can cause leakage, splashing or damage
6. Be careful with fingers and hands while lid is being opened and closed
7. Do not open lid further than its natural open position to prevent stress or cracking of the hinges
8. Do not stretch the silicone gasket
9. Do not Autoclave with lid on base
10. Do not use in a washer-disinfector with lid on base
11. Do not wash container with a hard brush. This can cause damage to the container
12. Do not expose the container to direct sunlight

#### Manual Cleaning:

1. Use a neutral (6.0 - 8.5) pH detergent
2. Remove all visible soil and contaminates
3. Immerge the container while cleaning to aid in the removal of contaminates and reduce splashing of detergent on personnel
4. Rinse thoroughly with clean water to remove detergent

Distributed by:



815 Vikings Parkway, Suite 100  
St. Paul, MN 55121 | USA

P : 1-888-229-2875 | 651-789-3939  
F : 1-888-229-1941 | 651-789-3979  
[www.summitmedicalusa.com](http://www.summitmedicalusa.com)