INTRUSAFE

LIFTING TRAY FOR THE XL TRANSPORT CONTAINER
PLATEAU DE LEVAGE POUR BOÎTE DE TRANSPORT XL
HERAUSNEHMBARER EINSATZ FÜR DEN XL TRANSPORTBEHÄLTER
BANDEJA ELEVADORA PARA EL RECIPIENTE DE TRANSPORTE XL
TABULEIRO DE ELEVAÇÃO PARA O RECIPIENTE DE TRANSPORTE XL
VASSOIO ESTRAIBILE PER IL CONTENITORE DI TRASPORTO XL
XL搬送用容器専用リフティングトレイ

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ENGLISH

Indications for Use

The Lifting Tray (IN-PIC-34) is an accessory for the XL Transport Container (IN-PIC-20/ IN-PIC-20-D). The Lifting Tray protects and organizes instruments and materials during transportation. The Lifting Tray can be used in an automated washer and can withstand repeated washing. **DO NOT** autoclave.

The Lifting Tray is designed to hold 1 *da Vinci Xi*[®] or 1 *da Vinci SP*[®] endoscope **OR** up to 8 *da Vinci Xi* or *SP EndoWrist*[®] instruments and associated accessories.

Material

The Lifting Tray is reusable and made of highly durable material.

Description

The Lifting Tray consists of a removable tray and basket that is meant to hold accessories and separate them from delicate instruments and/or scopes.

Lifting Tray Use

1. Set XL Transport Container on flat surface
2. Nest Lifting Tray in bottom of XL Transport Container
3. Place instruments or scope in Lifting Tray
 - A. FOR SINGLE ENDOSCOPE CONFIGURATION**

Place *da Vinci Xi* endoscope control head diagonally, **OR** place *da Vinci SP* endoscope control head horizontally on the corresponding section of the Lifting Tray and wrap cord loosely in a clockwise direction.
 - B. FOR MULTIPLE INSTRUMENT CONFIGURATION**

Place *da Vinci EndoWrist* instruments horizontally in the Lifting Tray alternating the direction of the control head each time. **Note:** *First control head should be placed in the top left corner before alternating instruments.*
4. Nest removable basket on center of Lifting Tray and load accessories into basket
5. Place lid on XL Transport Container and engage 4 silicone latches
6. Transport the XL Transport Container and its contents
7. Disengage XL Transport Container latches and open lid
8. Remove Lifting Tray and its contents
9. Process instruments/materials per the instruments/materials IFU

Warnings

1. Use handles to remove Lifting Tray from XL Transport Container
2. **DO NOT** Autoclave
3. Avoid dropping as this may cause damage to the instruments/materials inside and/or Lifting Tray
4. Clean and inspect prior to first use and after each use
5. Lifting Tray may be reused until mechanical failure occurs

Examples of mechanical failure include:

- Broken or cracked corners
- Material degradation

Disclaimer: Lifting Tray is warranted against defects in materials or workmanship for 1 year

Manual Cleaning

1. Use a neutral (6.0 – 8.5) pH detergent
2. Remove all visible soil and contaminants using a soft brush
3. Rinse thoroughly with clean water to remove detergent
4. Dry completely before storage

Automated Washer Cleaning

1. Lifting Tray is safe for use in automated washers and can withstand up to 250°F (121°C)

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da Vinci®, EndoWrist®, Xi®, and SP® are all registered trademarks of Intuitive Surgical, Inc.
InstruSafe® is a registered trademark of Summit Medical, an Innovia Medical Company.
MDSS GmbH