



# INSTRUSAFE

CARE + MAINTENANCE PRODUCTS

## Channel Cleaning Brushes

The Channel Cleaning Brushes are intended to remove surgical residue from endoscopic devices after surgical procedures. The Instructions for Use provided with the product state the following:

1. Advance the brush through the endoscope channel in short strokes to avoid kinking
2. **DO NOT** rotate the brush inside the endoscope channel
3. Inspect the brush between uses and replace when worn, frayed, bent or otherwise damaged
4. This inspection is the responsibility of the user
5. Worn bristles are ineffective in cleaning and damaged bristles may damage endoscope

The brushes must be cleaned thoroughly and may be ultrasonically cleaned to remove debris. If not thoroughly cleaned, sterilization may be compromised.

### Symbol Reference Key

	Consult Instructions For Use
	Quantity
	Catalog Number
	Lot Number
	Date of Manufacture
	Non-Sterile
	For Us Audiences Only

Distributed by:



815 Vikings Parkway, Suite 100  
St. Paul, MN 55121 | USA

P : 1-888-229-2875 | 651-789-3939  
F : 1-888-229-1941 | 651-789-3979

[www.summitmedicalusa.com](http://www.summitmedicalusa.com)